

## PATIENT INSTRUCTIONS

### BELOW KNEE AMPUTEE

- Only use the prosthesis under the direction of physical therapy until trained in use.
- Always use a walker.
- Make sure pin at the end of the liner engages into bottom of socket...you will hear 'clicks' as it goes into socket.
- Remove leg frequently the first few weeks to monitor skin condition.
- Clean rubber liner every night w/ antibacterial soap and water and let it dry
- Alternate use of liners
- Add soft socks over the rubber liner if the stump shrinks and the socket is loose. Add as many socks as needed to keep the socket snug.
- If patient complains of pain in stump, make sure patient has enough socks on and if that does not alleviate the problem, remove device and call MD Labs.
- Always use a shoe with the prosthesis.
- Stop using the device anytime there is a skin problem, and call MD LABS @ 1-888-MDLABS-1
- If you have any questions please call MD Labs.